


# CoreFlow<sup>®</sup>

URODYNAMIC INNOVATION FROM PROSTALUND




**CoreFlow<sup>®</sup> - Soft Stent**  
A new device which temporarily  
resolve obstruction of the prostatic  
urethra.



**Bring this information sheet with you if you are in contact with healthcare.**

The product is relatively new on the market. Therefore, it is important to provide information to healthcare personnel who have not yet used the product.

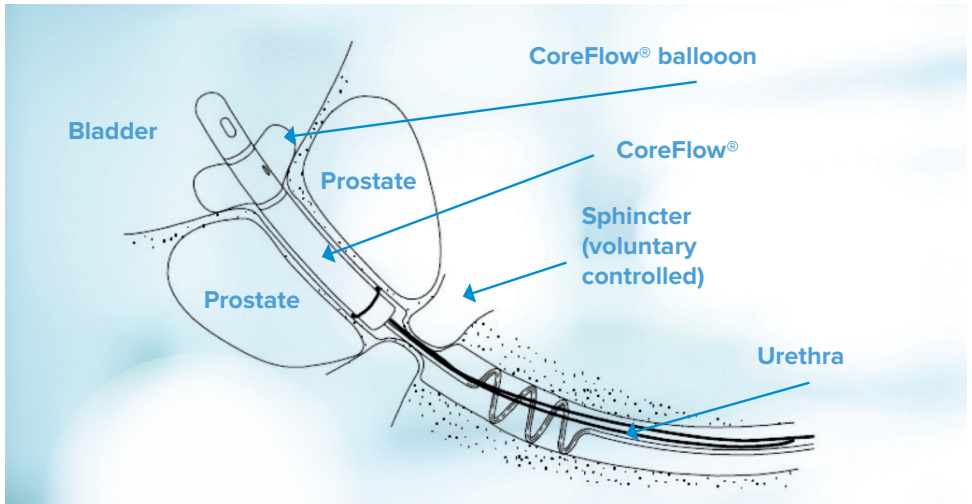


**CoreFlow® - Soft Stent is a new device which temporarily resolve obstruction of the prostatic urethra.**

A white tube and a blue thread, both parts of CoreFlow®, runs from the urethral opening on the penis.

The white tube (which your doctor made a couple of knots on) has been used to inflate the CoreFlow® balloon with liquid. The balloon anchors the CoreFlow® in place in the bladder to keep it from migrating.

When CoreFlow® shall be removed, the white tube is cut, the liquid flows from the balloon and CoreFlow® can be removed.


## CoreFlow® – Soft Stent

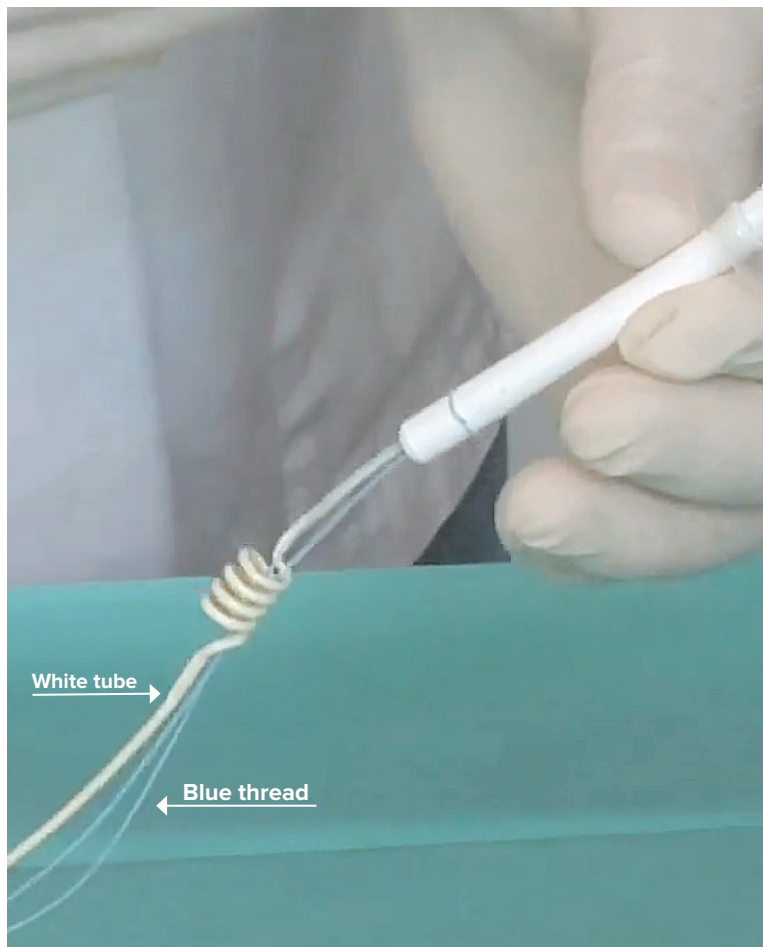

If CoreFlow® would migrate into the bladder, it is usually easy for the patient to adjust by application of slight traction on the blue thread.

If you also have difficulty urinating on your own with CoreFlow®, you can empty the bladder by pulling the blue thread. The urine flow stops when you release the thread. If you still cannot empty your bladder, contact your healthcare provider.

More information about CoreFlow® - Soft Stent can be found at [www.coreflow-softstent.com](http://www.coreflow-softstent.com)

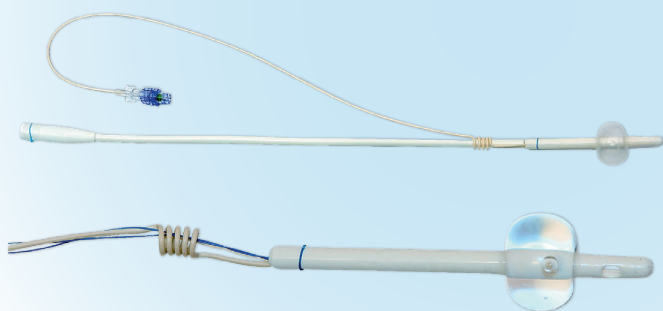

# CoreFlow<sup>®</sup>

URODYNAMIC INNOVATION FROM PROSTALUND

M-2020-048EN/1



ProstaLund AB | Scheelevägen 19 | SE-223 63 Lund, Sweden |  
[www.coreflow-softstent.com](http://www.coreflow-softstent.com) | [info@prostalund.com](mailto:info@prostalund.com)