
INSTRUCTION FOR USE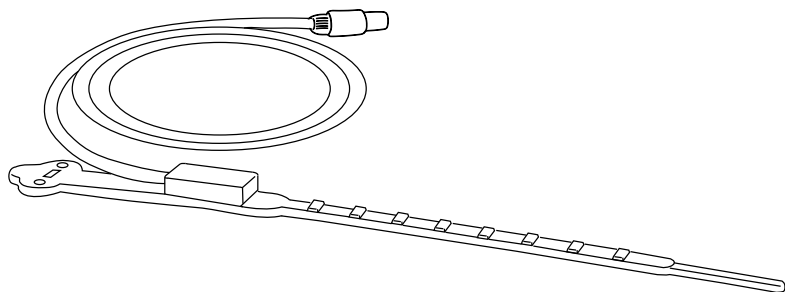

PENIS SAFETY PROBE INSTRUCTIONS FOR USE ENGLISH

DEVICE DESCRIPTION

The Penis Safety Probe records the temperature at the base of the penis during CoreTherm® treatment, see CoreTherm® IFU L-2013-009 / CoreTherm® Eagle IFU L-011.

The Penis Safety Probe contains a temperature sensor and an electronic chip. The electronic chip contains a unique serial number to identify the specific Penis Safety Probe in use.

The probe has a colour coded (grey) connector with a hardware key that prevents incorrect attachment of the probe to the control unit.

INTENDED PURPOSE

Intended use

The Penis Safety Probe is used with the ProstaLund CoreTherm® device. It records the temperature at the base of the penis during treatment.

The CoreTherm® with accessories is intended for treatment of lower urinary tract symptoms (LUTS) due to benign prostatic enlargement (BPE) using microwave thermotherapy by, or under the supervision of, a trained medical practitioner.

Intended patient population

Adult male patients with BPE.

Intended user profile

To be used by, or under supervision of, a trained medical practitioner.

Intended use environment

Hospitals and health care centers.

INDICATIONS

See CoreTherm® IFU L-2013-009 / CoreTherm® Eagle IFU L-011 for indications to CoreTherm® treatment.

CONTRAINDICATIONS

No known contraindications with the Penis Safety Probe.

See CoreTherm® IFU L-2013-009 / CoreTherm® Eagle IFU L-011 for contraindications for CoreTherm® treatment.

WARNINGS

There is a risk of complications if the Penis Safety Probe is not handled carefully and correctly. The risk of complications can be minimized by using the product as described in this instruction for use.

Patient injury

Exchange any item of damaged equipment for a fully operational, undamaged item before initiating the treatment. The use of damaged equipment may injure the patient.

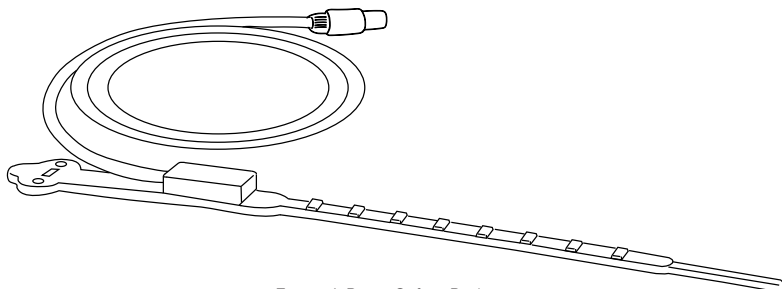
Figure 1. Penis Safety Probe.

Excessive temperature at the penoscrotal angle

To avoid the risk of burn injuries to the penis or urethra, the penile temperature should not exceed the temperature safety limit during treatment. Regularly monitor the penile temperature and check the position of the Penis Safety Probe during treatment.

See CoreTherm® IFU L-2013-009/ CoreTherm® Eagle IFU L-011 for detailed information related to the penile temperature limit and monitoring of the penile temperature during treatment.

PRECAUTIONS

- Keep the probe rolled in loose loops in the instrument case to protect the internal wires.
- The Penis Safety Probe is delivered unsterile. The probe needs to be cleaned and disinfected before use.
- When the probe is applied to the patient, check that the penile temperature is normal before starting the treatment. See CoreTherm® IFU L-2013-009/ CoreTherm® Eagle IFU L-011 for detailed information related to typical temperature for the Penis Safety Probe pre-treatment.
- Recommended reuse life for the Penis Safety Probes is 10 times.

DIRECTIONS FOR USE

The Penis Safety Probe can be reused provided that the instructions given in this section are followed. Since the Penis Safety Probe is a delicate instrument, it is very important that it is handled with care.

Prior to treatment

1. Inspect the Penis Safety Probe for cracks or any sign of physical damage. Exchange any item of damaged equipment for a fully operational, undamaged item before starting the treatment.
2. The Penis Safety Probe can be reused after cleaning and disinfecting and is supplied in non-sterile packaging.

Treatment

1. Place the Penis Safety Probe around the base of the penis, with the temperature sensor against the urethra and at the penoscrotal angle beneath the penis, see Fig 2. Fasten the strap and ensure that it is properly connected.

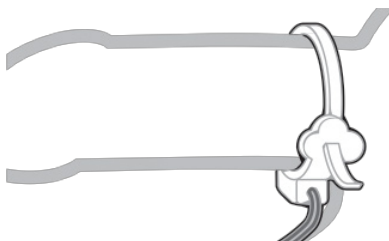
Figure 2. Attachment of Penis Safety Probe.

Connection of the Penis Safety Probe to the CoreTherm® device

Connect the Penis Safety Probe (SP) to the grey socket on the CoreTherm® device as illustrated in Figure 3 for CoreTherm® / SE and Figure 4 for CoreTherm® Eagle.

NOTE! To ensure that the connection is properly made, listen to an audible click during connection.

Figure 3. Connecting the Penis Safety Probe to CoreTherm® SE.

Figure 4. Connecting the Penis Safety Probe to CoreTherm® Eagle.

After treatment

Disconnect the Penis Safety Probe from the grey socket of the control unit by pulling it outwards.

Remove the Penis Safety Probe from the base of the patient's penis.

CLEANING

The Penis Safety Probe should be cleaned with one of the suggested agents:

COMMERCIAL NAME	ACTIVE COMPONENT
Hygisan	Alkali and surfactants
0,1 M NaOH	NaOH
K4 Sumazon Neste Dip	Enzymes
Extran AP 41	Enzymes
Klenzyme	Enzymes

1. Prepare the detergent according to the manufacturer's instructions in a container large enough to lower the probes.
2. Use a soft brush with synthetic fibers. A small pipe brush can be used for the narrow areas.
3. Clean the probe with the brushes for 4 minutes while immersed.

NOTE: Special attention should be paid to hard-to-reach areas, such as the coupling, and make sure to clean these areas particularly well.

4. The probe should then be rinsed under running tap water for three minutes to ensure that all remaining detergent is flushed off.

NOTE: Particular care should be taken to rinse off hard-to-reach areas.

5. The probe should then be gently wiped.

NOTE: Particular attention should be paid to the connector to the control unit if any fluid remains in that area. Hold the plastic part of the connector and shake it into the air to remove any drops from the inside of the connector.

NOTE: After cleaning, make sure that no blood residue is left in the connector or in any other areas.

DISINFECTION

The probe shall be disinfected using a 2-propanol disinfectant agent.

1. Prepare a regular shower/spray bottle with the disinfectant according to the manufacturer's instructions.
2. Carefully spray the probe with the disinfectant at a distance of 20 cm.
3. Allow the disinfectant to act on the probe for 10 minutes.
4. Wipe the safety probe gently including the connector of the probe.

CARE AND STORAGE

- The probe must be stored in a clean and dry place with temperatures between 5°C to 40°C, and with a humidity between 30% and 75% RH.
- Avoid exposing the units to direct sunlight.
- Keep the probe rolled in loose loops in the instrument case to protect the internal wires.
- The connector is permanently attached to the probe cable. Do not remove it. Be sure to keep the connector dry at all times.

REUSE

Recommended reuse life for the Penis Safety Probes is 10 times, after which it should be disposed.

SAFE DISPOSALS

The Penis Safety Probe shall be disposed according to the local regulations applying to hazardous biological substances.

REPORTABILITY

Any serious incident that has occurred in relation to medical device shall be reported to ProstaLund AB and to your national competent authority.

SYMBOLS

Type BF applied part

Read instructions for use

Manufacturer

Manufacturing date, YYYY-MM

Range of humidity

Temperature limits

Article number

Serial number

Medical Device - Indicates the item is a medical device

ORDERING INFORMATION

Device	Article Number
Penis safety probe	SP808601

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