
INSTRUCTION FOR USE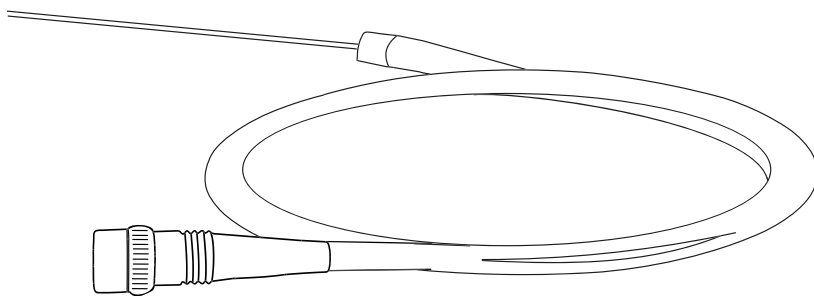

DEVICE DESCRIPTION

The Microwave Antenna (AN) directs the microwave radiation into the prostate tissue to heat the gland during CoreTherm® treatment, see CoreTherm® IFU L-2013-009 / CoreTherm® Eagle IFU L-011.

- The Microwave Antenna is delivered unsterile and can be reused 10 times after cleaning and disinfecting.
- The Microwave Antenna is inserted into the CoreTherm® Catheter.
- The Microwave Antenna has a Luer fitting with an integrated Luer-lock that secures it to the CoreTherm® Catheter.

The Microwave Antenna is available in two different versions, Short and Standard, see Fig 1. The certain prostate dimensions of the patient determine which version should be used.

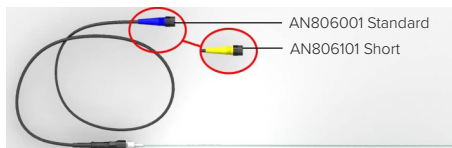
Figure 1. The short connector (yellow) and the standard connector (blue).

Microwave Antenna Short (yellow)

- Can be used with prostate lengths between 25-34 mm and/or prostate weights between 20-30 g.

Microwave Antenna Standard (blue)

- Can be used with prostate lengths from ≥ 35 mm and prostate weights ≥ 50 g.

Microwave Antenna Short and Standard

- Both Short and Standard Microwave Antenna can be used with prostate lengths ≥ 35 mm and prostate weights between 31-49 g.

		Prostate length	
		≥ 35 mm	25-34 mm
Prostate weight	≥ 50 g	Standard (blue)	Short (yellow)
	31-49 g	Standard (blue) Short (yellow)	Short (yellow)
	20-30 g	Short (yellow)	Short (yellow)

Valid prostate dimensions for treatment are prostate weights ≥ 20 g and prostate lengths ≥ 25 mm. Patients with prostate dimensions below these values should not be treated with CoreTherm®.

INTENDED PURPOSE

Intended use

The Microwave Antenna is used with the ProstaLund CoreTherm® device. It directs the microwave radiation into the prostate tissue to heat the gland.

The CoreTherm® with accessories is intended for treatment of lower urinary tract symptoms (LUTS) due to benign prostatic enlargement (BPE) using microwave thermotherapy by, or under the supervision of, a trained medical practitioner.

Intended patient population

Adult male patients with BPE.

Intended user profile

To be used by, or under supervision of a trained medical practitioner.

Intended use environment

Hospitals and health care centers.

INDICATIONS

See CoreTherm® IFU, L-2013-009 / CoreTherm® Eagle IFU L-011 for indications to CoreTherm® treatment.

CONTRAINDICATIONS

No known contraindications with the Microwave Antenna.

See CoreTherm® IFU L-2013-009 / CoreTherm® Eagle IFU L-011 for Contraindications for CoreTherm® treatment.

WARNINGS

There is a risk of complications if the Microwave Antenna is not handled carefully and correctly. The risk of complications can be minimised by using the product as described in this instruction for use.

Patient injury

- Exchange any item of damaged equipment for a fully operational, undamaged item before initiating the treatment. The use of damaged equipment may injure the patient.
- When preparing the patient for treatment, always check the front face and the cable of the Microwave Antenna for cracks or any sign of physical damage, as this may impair the performance and safety of the equipment.
- If the reflected microwave power cannot be reduced or is difficult to reduce by using the Stub Tuner, this may indicate that the Microwave Antenna is defective and must not be used.

Danger of microwave radiation

- Microwave power must not be emitted under any circumstances when the CoreTherm® Catheter and/or the Microwave Antenna are not correctly positioned within the body, as this may cause injuries to the patient and/or the operator.

Insufficient cleaning/disinfection

- Ensure that the Microwave Antenna is properly cleaned and disinfected in accordance with the instructions given in the section **CLEANING AND DISINFECTION**, otherwise infection may result.

PRECAUTIONS

- Do not bend the Microwave Antenna to a radius less than 60 mm.
- Do not bend the radiating part of the antenna (the most distal part of the antenna).

DIRECTIONS FOR USE

The Microwave Antenna can be reused provided that the instructions given in this section are followed. Since the Microwave Antenna is a delicate instrument, it is very important that it is handled with care. Recommended reuse life of the antenna is 10 times.

Prior to treatment

1. Check the front face and the cable of the antenna for cracks or any signs of physical damage, as this may impair the performance and safety of the equipment. Exchange any item of damaged equipment for a fully operational, undamaged item before starting the treatment.
2. The Microwave Antenna can be reused after cleaning and disinfecting and is supplied in non-sterile packaging.

Treatment

1. Insert the disinfected Microwave Antenna into the CoreTherm® Catheter and secure the antenna by turning the Luer-lock in the clockwise direction, see Fig 2. The same procedure applies for both antenna versions.

NOTE: Do not lubricate the Microwave Antenna.

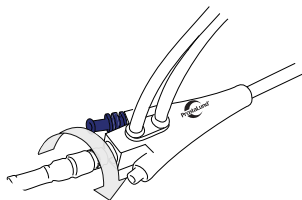
Figure 2. Securing the antenna in the CoreTherm® Catheter.

2. **CoreTherm®/SE:** Connect the Microwave Antenna (1) to the connector (2) on the left side of the CoreTherm® device (3), see figure 3.

CoreTherm® Eagle: Connect the Microwave Antenna to the connector on the left side of the CoreTherm® device, see yellow arrow in figure 4.

Rotate the connector 90 degrees to lock it in place. For further instructions please read the CoreTherm® IFU L-2013-009 / CoreTherm® Eagle L-011.

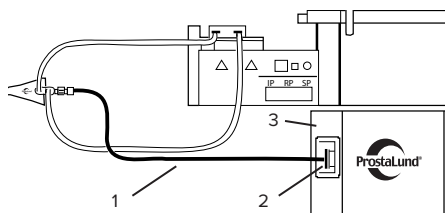
Figure 3. Connecting the Microwave Antenna to CoreTherm® / SE.

Figure 4. Connecting the Microwave Antenna to CoreTherm® Eagle.

After treatment

1. Disconnect the Microwave Antenna from the connector on the left side of the CoreTherm® device by rotating it 90° counterclockwise and pulling it outwards.
2. Unlock the Microwave Antenna from the CoreTherm® Catheter as follow:
 - Hold the Luer-lock on the antenna in one hand.
 - Hold the Luer part of the CoreTherm® Catheter in the other hand.
 - Gently turn the Luer-lock on the antenna in the counterclockwise direction.

- When the Luer-lock is open, withdraw the Microwave Antenna from the CoreTherm® Catheter.

CLEANING AND DISINFECTION

Before each treatment the Microwave Antenna needs to be cleaned and disinfected. Clean the antenna by gently wiping it with a damp cloth, using a soap and water solution. Then gently wipe the antenna with a cloth dampened in 70 % alcohol solution.

CARE AND STORAGE

The Microwave Antenna is a delicate instrument and should be handled with care.

- Do not bend the antenna to a radius less than 60 mm.
- Do not bend the radiating part of the antenna (the most distal part of the antenna).
- Always store the antenna in its instrument case.
- The antenna must be stored in a clean and dry place with temperature between 5°C to 40°C, and humidity between 30% and 75% RH.
- Avoid exposing the units to direct sunlight.

REUSE

Recommended lifetime of the antenna is 10 treatments after which it should be disposed.

SAFE DISPOSAL

The antenna shall be disposed according to the local regulations applying to hazardous biological substances.

REPORTABILITY

Any serious incident that has occurred in relation to the medical device shall be reported to ProstaLund AB and the authority having jurisdiction in your locale.

SYMBOLS

Type BF applied part

Read instructions for use

Manufacturer

Manufacturing date, YYYY-MM

Non-ionizing radiation

Range of humidity

Temperature limits

Article number

Serial number

Medical Device - Indicates the item is a medical device

ORDERING INFORMATION	
Device	Article Number
Microwave Antenna, Standard	AN806001
Microwave Antenna, Short	AN806101

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