
INSTRUCTION FOR USE

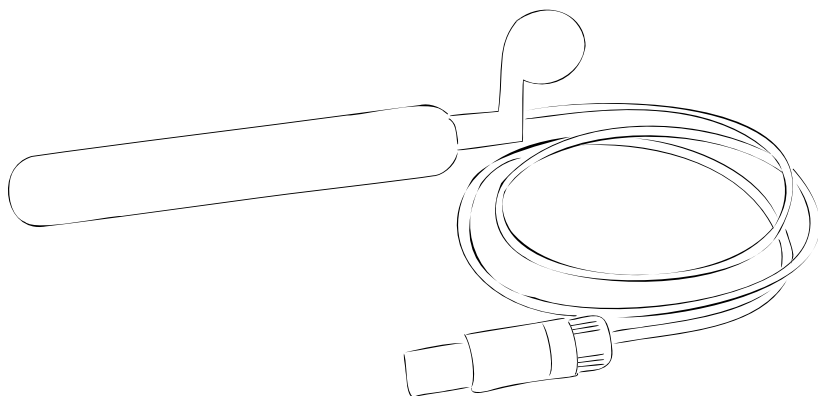RP809601

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Tel.: +46 (0)46 12 09 08

E-mail: info@prostalund.comwww.prostalund.se

RECTAL TEMPERATURE PROBE INSTRUCTIONS FOR USE ENGLISH

DEVICE DESCRIPTION

The Rectal Temperature Probe monitors the temperature in the patient's rectum during treatment see CoreTherm® IFU L-2013-009 / CoreTherm® Eagle IFU L-011.

The Rectal Temperature Probe contains three temperature sensors. The sensors are located at the tip on the anterior side of the probe.

The Rectal Temperature Probe has a color coded (blue) control unit connector. The connector has a hardware key that prevents the incorrect attachment of the probe to the Control Unit of the CoreTherm® device.

- An electronic chip in the connector contains a unique serial number to identify the specific Rectal Temperature Probe in use for the software.

INTENDED PURPOSE

Intended use

The Rectal Temperature Probe is used with the CoreTherm® device. It records the temperature in the patient's rectum during CoreTherm® treatment.

The CoreTherm® with accessories is intended for treatment of lower urinary tract symptoms (LUTS) due to benign prostatic enlargement (BPE) using microwave thermotherapy by, or under the supervision of, a trained medical practitioner.

Intended patient population

Adult male patients with benign prostatic enlargement (BPE) with a prostate weight of ≥ 20 g and prostate length ≥ 25 mm.

Intended user profile

To be used by, or under supervision of, a trained medical practitioner.

Intended use environment

Hospitals and health care centers.

INDICATIONS

See CoreTherm® IFU, L-2013-009/ CoreTherm® Eagle IFU L-011 for indications for CoreTherm® treatment.

CONTRAINDICATIONS

No known contraindications with the rectal temperature probe.

See CoreTherm® IFU L-2013-009/ CoreTherm® Eagle IFU L-011 for contraindications for CoreTherm® treatment.

WARNINGS

There is a risk for complications if the Rectal Temperature Probe is not handled carefully and correctly. The risk for complications can be minimized by using the product as described in this instruction for use.

Patient injury

Exchange any item of damaged equipment for a fully operational, undamaged item before initiating the treatment. The use of damaged equipment may injure the patient.

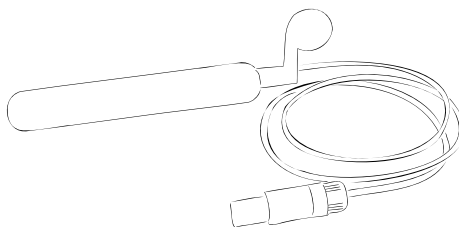
Figure 1. Rectal Temperature Probe.

Excessive Rectal Temperature

To avoid the risk of fistulas, the rectal temperature should not exceed temperature safety limits during treatment. You must check the position of the probe regularly throughout the treatment. See CoreTherm® IFU L-2013-009/ CoreTherm® Eagle IFU L-011 for detailed information related to rectal temperature limits and monitoring of the rectal temperature during treatment.

PRECAUTIONS

- Keep the probe, (with cable rolled in loose loops), in the instrument case to protect the internal wires.
- The Rectal Temperature Probe is delivered unsterile. The probe needs to be cleaned and disinfected before use.
- When the probe is applied to the patient, check that the patient's temperatures are normal before starting the treatment. See CoreTherm® IFU L-2013-009/ CoreTherm® Eagle IFU L-011 for detailed information related to typical temperatures for the Rectal Temperature Probe pre-treatment.
- Recommended reuse life for the temperature probes are 10 times.

DIRECTIONS FOR USE

The Rectal Temperature Probe can be reused provided that the instructions given in this section are followed. Since the Rectal Temperature Probe is a delicate instrument, it is very important that it is handled with care.

Prior to treatment

1. Inspect the Rectal Temperature Probe for cracks or any sign of physical damage. Exchange any item of damaged equipment for a fully operational, undamaged item before starting the treatment.
2. The Rectal Temperature Probe can be reused after cleaning and disinfecting and is supplied in non-sterile packaging.

Treatment

1. Position the patient to receive the Rectal Temperature Probe. The patient can remain in the supine position with one leg drawn up or lie on his left side. Have a rolled-up towel available to stabilize the rectal probe.

2. Cover the Rectal Temperature Probe with a condom and apply a waterbased lubricant gel to the outer surface of the condom. Then, with the handle of the probe pointing vertically upward, carefully introduce the probe into the rectum.

NOTE: Do not apply gel to the inside of the condom.

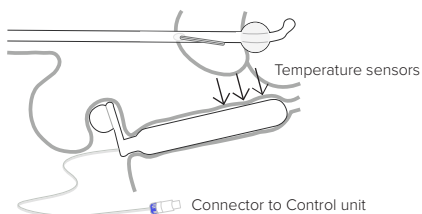
Figure 2. Correct position of the Rectal Temperature Probe.

3. Place the rolled-up hand towel under the patient to prevent ejection of the Rectal Temperature Probe when the patient is laid flat.
4. Connect the Rectal Temperature Probe to the blue socket as illustrated below in Fig 3 (yellow arrow) for CoreTherm® Eagle and Fig 4 (white arrow) for CoreTherm® SE. An audible click will be heard when the connection is properly made.

Figure 3. Connecting the Rectal Temperature Probe to CoreTherm® Eagle.

Figure 4. Connecting the Rectal Temperature Probe to CoreTherm® / SE.

After treatment

Disconnect the Rectal Temperature Probe from the blue socket of the control unit by pulling it outwards.

Carefully withdraw the Rectal Temperature Probe from the patient.

CLEANING

The Rectal Temperature Probe should be cleaned with one of the suggested agents:

COMMERCIAL NAME	ACTIVE COMPONENT
Hygisan	Alkali and surfactants
0,1 M NaOH	NaOH
K4 Sumazon Neste Dip	Enzymes
Extran AP 41	Enzymes
Klenzyme	Enzymes

1. Prepare the detergent according to the manufacturer's instructions in a container large enough to lower the probes.
2. Use a soft brush with synthetic fibers. A small pipe brush can be used for the narrower areas.
3. Clean the probe with the brushes for 4 minutes while immersed.

NOTE: Special attention should be paid to hard-to-reach areas, such as the coupling, and make sure to clean these areas particularly well.

4. The probe should then be rinsed under running tap water for three minutes to ensure that all remaining detergent is flushed off.

NOTE: Particular care should be taken to rinse off hard-to-reach areas.

5. The probe should then be gently wiped.
6. Wipe the probe gently against the tip, including the connector.

NOTE: Particular attention should be paid to the connector to the control unit if any fluid remains in that area. Hold the plastic part of the connector and shake it into the air to remove any drops from the inside of the connector.

NOTE: After cleaning, make sure that no blood residue is left in connector or in any other areas.

DISINFECTION

The probe shall be disinfected using a 2-propanol disinfectant agent.

1. Prepare a regular shower/spray bottle with the disinfectant according to the manufacturer's instructions.
2. Carefully spray the probe with the disinfectant at a distance of 20 cm.
3. Allow the disinfectant to act on the probe for 10 minutes.
4. Wipe the probe, by wiping gently against the tip, including the connector of the probe.

CARE AND STORAGE

- The probe must be stored in a clean and dry place with temperature between 5°C to 40°C and 30% - 75% R-H.
- Avoid exposing the unit to direct sunlight.
- Keep the probe, (with cable rolled in loose loops), in the instrument case to protect the internal wires.
- The connector is permanently attached to the probe cable. Do not remove it. Be sure to keep the connector dry at all times.

REUSE

Recommended lifetime of the Rectal Temperature Probe is 10 treatments after which it should be disposed.

SAFE DISPOSAL

The Rectal Temperature Probe shall be disposed according to the local regulations applying to hazardous biological substances.

REPORTABILITY

Any serious incident that has occurred in relation to the device should be reported to ProstaLund AB and to your national competent authority.

SYMBOLS

Type BF applied part

Read instructions for use

Manufacturer

Manufacturing date, YYYY-MM

Range of humidity

Temperature limits

Article number

Serial number

Medical Device - Indicates the item is
a medical device

ORDERING INFORMATION

Device	Article Number
Rectal temperature probe	RP809601

CONTACT INFORMATION

ProstaLund AB
 Scheelevägen 19
 SE-223 63 Lund
 Tel: +46 (0)46 12 09 08
 E-mail: info@prostalund.com
www.prostalund.se

RP809601